- 1. Please click on a link below to **download the appropriate installer** for the MIM Referring software.
 - For PC/Windows users: https://downloads.mimsoftware.com/MIM7.0.6_buildKB13-05.exe
 - For Mac users: https://downloads.mimsoftware.com/MIM7.0.6_buildKB13-05.dmg
- 2. Install the application.
 - For PC/Windows users: click Run as Administrator to begin the download and follow the installation instructions; see screenshot below.

🖊 🛃 🗖 🖛 D	ownload	ds									- C	x t
File Home	Share	View										^ ?
Pin to Quick Copy access	Paste	X Cut ‱ Copy path ₽ Paste shortcut	Move Copy to * to *	ename New folder	new 🚹 Easy	item • access •	Properties	🔛 Open 👻 🃝 Edit 🊱 History	Select all Select none			
Clipboard			Organize		New		0	pen	Select			
\leftrightarrow \rightarrow \checkmark \bigstar This PC \Rightarrow Downloads \checkmark \circlearrowright \circlearrowright Search Downloads												
🗙 🔮 Quick access		Name						Date	modified	Туре	Size	^
Desktop	*	✓ Today (1) —										
Downloads MIM7.0.6_buildKB13-05 (2).exe						9/30	/2021 12:23 PM	Application	4	02,181 KB		
Documents	*	✓ Yesterday (6)										_

• For Mac users: click on the downloaded .dmg file and then click on the PKG file to begin the installation.

(> ()	
Favorites Name Size Kind	Date Added
Obwnloads AIM/0.0.6_build/KB13-05.dmg 432.1 MB Disk Image	Today at 7:42 AM

3. Enter the password korey (all lowercase) when prompted; see screenshot below.

Image: MIM 7.0.6 build KB13-05 Install/Upgrade Image: Comparison of the state of th		×
Enter password for the encrypted file: MIM 7.0.6.KB13-05\setup.exe OK Cancel Extracting MIM 7.0.6.KB13-05\setup.exe Installation progress		
Install	Cancel	



PP-FT-US-0008 04/2022 © Lilly USA, LLC 2022. All rights reserved.

TAUVID[™] is a trademark owned or licensed by Eli Lilly and Company, its subsidiaries, or affiliates. Copyright © 2022 MIM Software Inc. All rights reserved.

 Once the installation completes, you will need to accept the end user license agreement and click Next; see screenshot below.



5. To configure the install, you can accept the **file install location** default choice or change it if desired; then click **Next**.





6. The next step asks if you'd like to **create a shortcut**; leave the selection at "All Users" and click Install; see screenshot below.



7. MIM will install.

Step 4 of 5: Installing	🍡 MIM Install Utility - MIM 7.0.6		-	×
Progress	Minimatell Utility - Mini 70.0	Step 4 of 5: Installing Progress 13% Copying cv/210 dll		×

8. Once finished installing, MIM will load after you click "Start MIM."



- **9.** For the licensing option, **select the top drop-down menu** and switch from "Fixed" to "**MIMCloud License**" and enter the log-in credentials provided below.
 - Username: TAUVID@TAUVIDReaderTraining.com
 - Password: Tauvid2021



PP-FT-US-0008 04/2022 © Lilly USA, LLC 2022. All rights reserved. TAUVID[™] is a trademark owned or licensed by Eli Lilly and Company, its subsidiaries, or affiliates. Copyright © 2022 MIM Software Inc. All rights reserved.



MIM Referring Software Setup and Log-in Instructions for the TAUVID Reader Training Website (cont.)

10. After entering the username and password listed in Step 9, **select "MIM Referring**" as your License Selection; then click **OK** to continue.

